

ReSound LiNX Quattro™

Confidence in the finer details

Discover the hearing solution
that suits you



ReSound GN
.....

ReSound LiNX Quattro™

Confidence in the moments that matter

Our newest complete collection of premium hearing aids is beyond compare and is certain to offer the perfect solution to fit your needs and lifestyle. Experience richer sound quality, a full spectrum of streaming, support and personalisation from anywhere, anytime, and the world's most advanced rechargeable solution.

Right down to the finer details, ReSound LiNX Quattro offers you a brilliant experience with Layers of Sound - and inspires confidence in the moments that matter most to you.



Brilliant sound experience




CLEARER SPEECH

Building on years of innovation, we continue to pioneer hearing aid technology that helps people hear better in their everyday lives. Our unique solution makes it easier for you to pick up speech and sounds from different directions in your environment. From the bustling street to the quiet theatre, you can focus on the conversation and still hear the sounds around you.

FULLER SOUNDS

There are now virtually no limits to your sound experience with ReSound LiNX Quattro, and you can hear it in the finer details. Both low and high-pitched sounds are natural and fuller. Quiet situations are truly quiet and louder sounds are comfortable, crisp and distortion-free. From the call of birds to the laughter of children and the notes in music, sounds are more accessible and enjoyable.

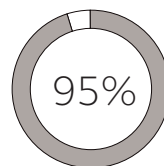

In a double-blind test, ReSound LiNX Quattro was preferred 95% of the time for music listening, compared to other premium hearing aids.

-  **ReSound LiNX Quattro**
-  **Other premium hearing aids**

RICHER EXPERIENCE

The whirring of the coffee machine in the morning and the music playing in the evening – with improved clarity and fullness of sounds – your favourite environments become richer.

The result is a brilliant sound experience.

TOP RATED IN BLIND TESTS

In blind listening tests, people could locate and understand speech better in noisy environments with ReSound LiNX Quattro. And ReSound LiNX Quattro was rated significantly better for sound quality for music listening.

Find out more about the brilliant sound experience at
[resound.com](https://www.resound.com)





Unrivalled full spectrum of streaming

MORE LAYERS OF SOUND, ACTION AND RHYTHM

When you answer the phone on the train to work, you can follow the conversation. As you watch TV at home, you pick up more of the action. And when you drive through the city, you hear real-time navigational instructions. ReSound LiNX Quattro provides you with an exceptional streaming experience from virtually any audio source. And with clearer, fuller and richer sounds, you can enjoy the finer details of speech, notes and tones – all streamed directly to your hearing aids.

Find out more about the streaming experience and the full compatibility list of smartphones at resound.com

BE CONFIDENT ON THE PHONE

Easily pick up phone calls with more clarity. Research shows that on average, you will be able to understand an additional 50% of the conversation when streaming to your hearing aids, compared to a regular phone call. And now you can stream directly from your iOS and Android™ devices* in stereo with no need for an intermediate device.

A CLEAR NUMBER ONE FOR STREAMING

When streaming directly from an iPhone, listeners rated the sound quality of ReSound LiNX Quattro 37% better on average compared to other premium hearing aids.

MORE ACTION ON TV

Hear the news, watch movies and hear the soundtrack with improved clarity at your own volume – while the rest of your family listens at theirs.

* From Android version 10 with Bluetooth ver 5.0 and the Android Streaming to Hearing Aids feature

ReSound Assist

Support and personalisation from anywhere

ReSound Assist gives you even more options to stay in touch with your hearing care professional and receive help and support that fits your schedule.

FACE-TO-FACE, ONLINE HEARING CARE WHEREVER, WHENEVER

ReSound Assist Live* and the ReSound Smart 3D™ app make it possible to have face-to-face fitting sessions from anywhere. Use the app to schedule a video call with your hearing care professional to fine-tune your hearing aids remotely.

* Please ask your hearing care professional for date of availability

ADJUSTMENTS ANYWHERE, ANYTIME

At home, at work, on vacation, ReSound Assist also lets you send requests from your ReSound Smart 3D app for fine-tuning and updates, straight from your hearing care professional.

And once you've tried them, you can send a satisfaction rating back to your hearing care professional to let them know you are satisfied with the new improvements.

Always receive the best hearing care experience with ReSound Assist and ReSound LiNX Quattro.


RESOUND ASSIST USERS ARE SAYING:

83% "It's easy to request assistance from the app"

89% "It's easy to apply new settings from the app"

77% "The new settings improved my hearing experience"



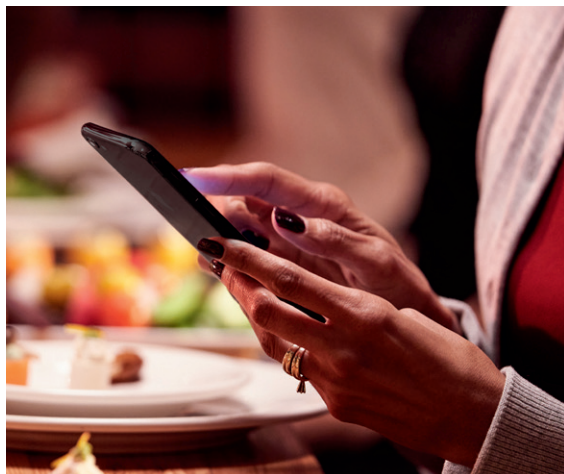

ReSound Smart 3D™ app

Optimise anytime, anywhere

ReSound LiNX Quattro hearing aids automatically adjust to your environment, but you can personalise the details of your hearing experience even further. With everything you need for discreet and quick adjustments, you are in control of how you experience the sounds around you.



Find out more about the ReSound Smart 3D app at
resound.com

Below are examples of features in the ReSound Smart 3D app.

SOUND ENHANCEMENTS

Access extra sound adjustments and get a fuller sound quality when adjusting bass, middle and treble.

BATTERY CHARGE STATUS

Track the battery status on each rechargeable hearing aid*, and get a notification when they need recharging.

FIND MY HEARING AIDS

If you can't find your hearing aids, you can check their last recorded position on a map.

RELIEF FROM TINNITUS

Experience relief and help alleviate the annoyance of tinnitus by playing soothing background sounds.

* Applies to Receiver-In-Ear rechargeable model

World's most advanced rechargeable solution



A hearing aid innovation, ReSound LiNX Quattro is the world's most advanced rechargeable hearing aid – with a lithium ion battery that is completely sealed for protection. With its small, discreet size and long-lasting battery, you get power that lasts more than a full day – without having to fiddle with any batteries. At night, simply place your hearing aids in the easy-to-use inductive charging case to prepare for the next day. From morning to night, you can be confident in your complete rechargeable solution.

FULL CONFIDENCE WITH A COMPLETE RECHARGE

Enjoy 24 hours of battery life – even when streaming television, music or other audio half of the time.

REFRESH WITH A QUICK RECHARGE

If you're on the go, traveling or if you forget to recharge overnight, you can still get long-lasting power with just a 30-minute recharge.



CHARGING TIME	HOURS OF BATTERY LIFE
3 hours =	30 24 ^{with 50% streaming time}
1 hour =	16
30 minutes =	8

Find out more about rechargeability at
resound.com

SLEEK AND FUNCTIONAL CHARGING CASE

Slipped easily into a pocket or travel bag, the small premium charging case keeps your hearing aids safe and powered – wherever you go.

Inductive charging pods make it secure and easy to place your hearing aids in the case for convenient charging. Besides being able to monitor the battery status in the ReSound Smart 3D app, an LED light on the hearing aid tells you that charging has begun. The case's built-in battery holds three full charges, and LED lights on the case indicate the power status of both the case and your hearing aids.



Small and perfectly designed

If you value a discreet hearing aid, the mini Receiver-In-Ear model is the right choice. It gives a cosmetically appealing look that is hard to spot. So small in size and sensitively styled that it's almost invisible.

Shown in actual size.

0 24,8 mm



Discover the hearing solution that suits you



World's most advanced rechargeable solution Receiver-in-Ear

Mini Receiver-in-Ear


Standard Receiver-in-Ear



Mini Behind-the-Ear

Standard Behind-the-Ear

Power Behind-the-Ear

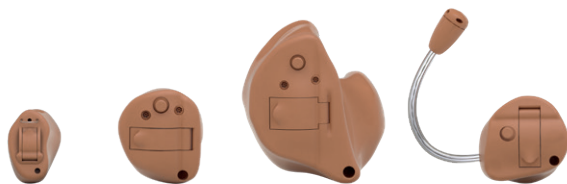

Completely-in-Canal

In-the-Canal

In-the-Ear

Mic-in-Helix

The same clearer, fuller and richer sound is available across all hearing aid models. ReSound LiNX Quattro is available in small Receiver-in-Ear models - including the highly attractive cosmetic mini Receiver-in-Ear and small and long-lasting rechargeable solution. It is moreover available in easy to handle Behind-the-Ear models and discreet custom-crafted models. The complete collection ensures there will be an option to suit your needs and lifestyle. And, with high-level performance, low-power consumption and direct streaming from both iOS and Android devices, you can enjoy the serenity of nature or the hustle and bustle of the city with complete confidence.

Colours



CUSTOM-CRAFTED MODELS



People with hearing loss are at the heart of everything we do because we know they're placing their trust in us. This is why we've not stopped pioneering innovative new technologies, with consistent quality, for over 150 years to help them feel more involved, connected and in control, transforming their lives through the power of sound. Available exclusively through audiologists and hearing care professionals.

ReSound is part of the GN Group – pioneering great sound from world-leading ReSound hearing aids to Jabra office headsets and sports headphones. Founded in 1869, employing over 5,000 people, and listed on NASDAQ OMX Copenhagen, GN makes life sound better.

Find out more about how Resound LiNX Quattro™ can provide confidence in the finer details at **resound.com**.

 facebook.com/resoundhearing

 twitter.com/resoundglobal

 linkedin.com/company/resound

Worldwide headquarters

GN Hearing A/S

Lautrupbjerg 7

DK-2750 Ballerup, Denmark

Tel.: +45 4575 1111

resound.com

© 2020 GN Hearing A/S. All rights reserved. ReSound is a trademark of GN Hearing A/S. Apple, the Apple logo, iPhone, iPad and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries. Android, Google Play and the Google Play logo are trademarks of Google LLC. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc.